

Fargo Safety & Warnings

Contraindications

Fargo is contraindicated when, in the medical judgment of the physician, use of such product may compromise the patient's condition.

This medical device is contraindicated for neonates, premature neonates and infants.

Potential Complications

Potential complications associated with the use of guiding catheters or with the endovascular procedures include but are not limited for:

- Access site complications - Ischemia
- Allergic reaction
- Neurological deficit
- Vessel perforation
- Stroke
- Aneurysm rupture
- Transient ischemic attack
- Death
- Vasospasm
- Embolism
- Vessel dissection
- Hematoma
- Vessel occlusion
- Hemorrhage
- Vessel rupture
- Infection
- Vessel thrombosis

Precautions for Use



Do not use if the pouch is open or damaged. These products are sterile when the packaging is not damaged.

- These products are intended for single use only. Do not reuse. Any reuse of the device cause a high risk of microbiological contamination for the patient as well as a risk of loss of the device characteristics.
- Do not resterilize.
- Store in a dry place at room temperature and away from light.
- Do not use the product after the expiry date.

- These products must be used by specialist physicians in interventional neuroradiology and / or in interventional radiology and / or in cardiology.

- Never advance or withdraw an intraluminal device against resistance until the cause of this resistance is determined by fluoroscopy.

Any forcing against the resistance could damage the catheter or bring about lesion in the patient.

- The instructions for use of products to be flushed through guiding catheters must be respected.

- Flush the guiding catheter during the procedure in order to avoid clotting.

- Respect the instructions for use of micro-catheters used into guiding catheters.