

Vasco Safety & Warnings

Contraindications

VASCO+ microcatheter is contraindicated when, in the medical judgment of the physician, use of such product may compromise the patient's condition. The VASCO+ microcatheter is contraindicated for neonates, premature neonates and infants.

Potential Complications

Potential complications include, but are not limited to:

- Puncture site hematoma,
- Pain and tenderness,
- Infection,
- Vessel or aneurysm perforation or dissection,
- Vasospasm,
- Haemorrhage,
- Thromboembolic episodes (Vascular thrombosis, embolism),
- Neurological deficits including stroke,
- Death.

Precautions for Use



- Do not use if the pouch is open or damaged. These products are sterile when the packaging is not damaged.
- VASCO+ microcatheter is intended for single use only. Do not reuse. Any reuse of the device cause a high risk of microbiological contamination for the patient as well as a risk of loss of the device characteristics.
- Do not resterilize.
- Store in a dry place at room temperature and away from light.
- Do not use the product after the expiry date.
- These devices must be used by specialist physicians in interventional neuroradiology and / or specialist physicians in interventional radiology.
- The instructions for use of products to be used with VASCO+ microcatheter must be respected.
- Never force an intravascular device against resistance without having first checked the cause by angiographic control. Forcing against resistance may damage the microcatheter or induce lesions in the patient.
- Control microcatheter infusion flow rate. If the flow rate decreases, determine the cause and correct it or replace the microcatheter.
- Refer to the labelling of devices used with VASCO+ and Gama to verify their compatibility.
- Refer to the instructions for use of other products used during the procedure.

Warnings

Caution: VASCO+ microcatheters are not DMSO agents compatible.